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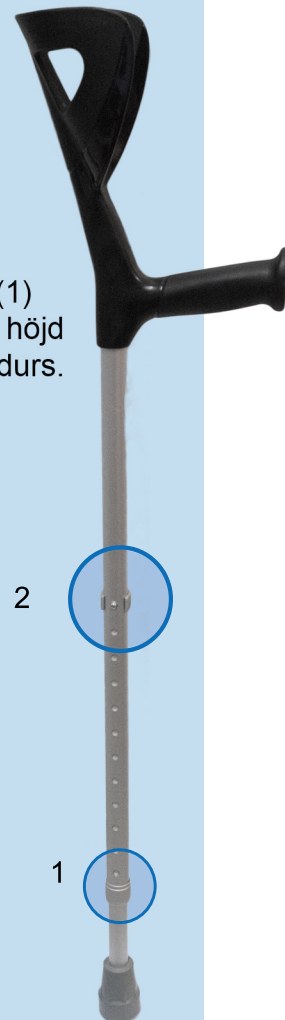
Crutch Tyke

340154 | 340154-JP
340154L | 340154L-JP
340154P | 340154P-JP

Improper use or adjustment of the product may lead to overstraining the product or the user. Always consult a professional before use to select the right type of device and adjustments for you.

Keep the product as close to your body as possible when moving to avoid accidents. Move forward calmly and in short steps to maintain balance.

The length of the crutch should be adjusted to suit the user. Before adjusting, loosen the silencer (1) by turning it counterclockwise. Remove the quick release clip (2), adjust the inner tube to the appropriate height and press the clip back into place. Tighten the silencer by turning clockwise.

Always make sure that the height adjustments are properly locked!

Be careful not to pinch your fingers or other body parts when handling the product.

WASHING and DRYING

The crutch should be cleaned with a mild detergent and a soft cloth. After cleaning, rinse it with clean water and then dry with a cloth. Check that no water remains inside the tube!

To remove stubborn stains, it is recommended to use grease-dissolving detergents, pH value 5-9, or 70% disinfectant. Washing temperature max. 80°C.

NOTE! Never use oily substances! Risk of slipping!

Do not use abrasive cleaning tools or substances, or strong solvents!

MAINTENANCE

The rubber plug should be checked and replaced if it is damaged or shows significant wear.

Remove the rubber plug by pulling it off the tube. Install the new rubber plug by pressing the tube all the way into the plug. Always make sure that the rubber plug is the correct size and model!

Also check the condition of the ice spike (if applicable). Make sure that the attachment is secure and the screws are tight enough. Check that the spike is sharp enough. A damaged or worn ice spike should be replaced with a new one.

A bent or damaged product must not be used. Do not attempt to repair the crutch by straightening it, as its structure may weaken and it may break.

It is recommended to use only original spare parts for repairs and maintenance. By using original spare parts, you ensure CE approval and the validity of the warranty.

2

1