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**Table of contents:**

**EN**

*\* Store the manual for future reference \**

**20**

**Bedside support, hex screw mounted 23**

654100 / 654120 / 654200 / 654220

**Bedside support, thumbscrew mounted 24**

654100S / 654120S / 654200S / 654220S

**Bedside support, screw mounted 25**

654100R / 654120R / 654100RJ / 654120RJ / 654200RJ / 654220RJ

**Bedside support for a motorized bed 26**

655120R / 655220R / 655120S / 655220S

**Bedside support, attached to the bottom 27-28**

654300R / 654320R / 654400R / 654420R

***Read this manual carefully and follow the assembly instructions. Careful assembly guarantees safe use.***

***The product installer must read the installation manual and watch the installation video. The installation does not require special skills and can be performed by the user.***

*Do not use this equipment without first reading and understanding this instruction manual. If you are unable to understand the warnings, cautions and instructions, contact a healthcare professional, dealer or technical personnel before attempting to install or use this equipment - otherwise, injury or damage may occur. Each individual should always consult with their physician or therapist to determine the best suited assistive aid for them and the proper adjustment and usage of it.*

**Watch the assembly video on YouTube:**

***\* Please note that the illustrations in this manual are only for guidance and may contain differences with the actual product.***

## Specifications:

**Intended use:** The bedside support is intended to be attached to the bed as an aid for adults to get themselves up. Bedside supports make it easier to get in and out of bed. The basic bedside supports are suitable for fixed wooden side or frame mattress beds. For motorized beds, there are separate bedside support models suitable for their requirements.

- Powder coated, chromed and/or galvanized steel pipe
- All models have stepless height adjustment, which can be tightened with either thumbscrews or hex screws
- Some of the models have a rotating upper support that turns in three different directions without tools
- Floor supports are available in two different heights
- Maximum user weight: 150 kg
- CE marked medical device

Meets the requirements of the standard SFS-EN ISO 60601-2-52 as applicable.

### ⚠ General safety instructions:

- **ENTRAPMENT HAZARD!** (see illustration on page 29)
- Avoid putting your fingers or other parts of the body in places where they might get pinched
- Operating temperature between -30°C and 40°C (a product that has been in a very hot or cold place should be handled carefully so as not to injure yourself)
- **NOTE! There must not be a carpet under the floor support!**
- Do not make structural adjustments or changes!
- Check the tightness of the mounting parts, screws, thumbscrews and/or hex screws at regular intervals. **A defective device must not be used. Don't try to fix it yourself but take it to a professional!**

Only original Tyke® spare parts may be used for repair and maintenance in order for the CE approval of the device to remain valid.

**Washing:** Keep the product clean of all dirt by wiping it with a damp cloth and a mild detergent to keep it in good shape. The product can also be disinfected.

⚠ **Never use oily substances! The surface of the product becomes slippery and the grip does not hold! Do not use harsh abrasive materials or bleach to avoid wear and tear and color fading.**

**PRODUCT EXPECTED LIFETIME\*:** 15 years (all models)

\* The expected life specified by the manufacturer is an estimate of the period of time during which the product is expected to function reliably and safely with proper use and maintenance. The calculation begins at the time of manufacture of the product and the service life depends on the operating conditions and frequency of use. The device can still be used after the period specified by the manufacturer has ended if a professional has inspected the device and determined that it is safe to use. The next inspection date must be determined in connection with the inspection. The manufacturer's recommendation for the inspection interval in such a case is max. 12 months.

## Hex screw mounted bedside supports

A 5 mm hex key is supplied with the product.

### Mounting:

Loosen the **hex screws 1-3** and **thumbscrew 4** to separate the side mounting part, upper support and floor support from each other.

Place the **side mounting part** on the side of the bed at a distance of about 60-80 cm from the headboard (recommended position; can also be installed according to the user's personal needs).

*The side part can be placed not only on the top edge of the side of the bed, but also on the bottom edge when turned the other way around, for example in the case of a frame mattress bed.*

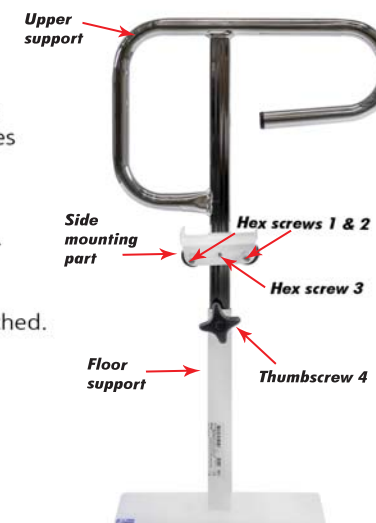
Fasten the side mounting part by tightening **hex screws 1 and 2**. Place the **floor support** in line with the opening of the side mounting part and place the **upper support** through the side mounting part and inside the floor support so that the longer end of the upper support faces the foot of the bed.

Adjust to a suitable height and tighten **hex screw 3** on the side mounting part and **thumbscrew 4** on the floor support.

Lastly, make sure that the product is firmly attached.  
-> The bedside support is ready for use.

**CHECK REGULARLY THE TIGHTNESS OF THE SCREWS.**

⚠ **NOTE!**  
**A safety limit marking (hole) is marked on the lower edge of the upper support (5cm). The upper support must be installed in such a way that the marking is not visible above the floor support.**

⚠ **NOTE! THERE MUST NOT BE A CARPET UNDER THE FLOOR SUPPORT!**

⚠ **MUST NOT BE MOUNTED ON MOTORIZED BEDS!**  
In unclear situations, ensure suitability!

## Thumbscrew mounted bedside supports

### Mounting:

Loosen the **thumbscrews 1-4** to separate the side mounting part, upper support and floor support from each other.

Place the **side mounting part** on the side of the bed at a distance of about 60-80 cm from the headboard (recommended position; can also be installed according to the user's personal needs).

*The side part can be placed not only on the top edge of the side of the bed, but also on the bottom edge when turned the other way around, for example in the case of a frame mattress bed.*

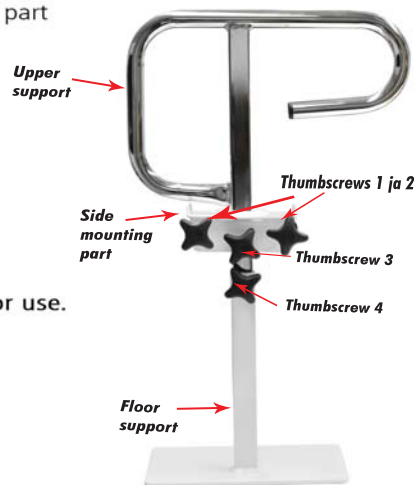
Fasten the side mounting part by tightening **thumbscrews 1 and 2**. Place the **floor support** in line with the opening of the side mounting part and place the **upper support** through the side mounting part and inside the floor support so that the longer end of the upper support faces the foot of the bed.

Adjust to a suitable height and tighten **thumbscrew 3** on the side mounting part and **thumbscrew 4** on the floor support.

Lastly, make sure that the product is firmly attached -> **The bedside support is ready for use.**

**CHECK REGULARLY THE TIGHTNESS OF THE SCREWS.**

**NOTE!**  
A safety limit marking (hole) is marked on the lower edge of the upper support (5cm). The upper support must be installed in such a way that the marking is not visible above the floor support.

**NOTE!** THERE MUST NOT BE A CARPET UNDER THE FLOOR SUPPORT!

**MUST NOT BE MOUNTED ON MOTORIZED BEDS!**  
In unclear situations, ensure suitability!

## Screw mounted bedside supports

Use eight (8) screws or through-bolts to attach the product, depending on the strength of the bed side. (NOTE! Not included in the product!)

### Mounting:

Loosen the **thumbscrew** to separate the side mounting part from the upper support. Place the **side mounting part** on the side of the bed at a distance of about 60-80 cm from the headboard (recommended position; can also be installed according to the user's personal needs). Attach the side mounting part with 8 screws or through-bolts depending on the strength of the side of the bed.

Place the **upper support** through the side mounting part so that the longer end of the upper support faces the foot of the bed. Adjust to a suitable height and tighten the thumbscrew on the side mounting part. Lastly, make sure that the product is firmly attached.  
-> **The bedside support is ready for use.**

**ATTACHMENT STRENGTH DEPENDS ON THE STRUCTURE OF THE WOODEN PARTS OF THE BED FRAME. IF POSSIBLE, IT IS RECOMMENDED TO ALWAYS USE A FLOOR SUPPORT.** The screw mounted bedside support is also available in combination with the floor support or the floor support can be purchased as an accessory.

654100RJ | 654120RJ | 654200RJ | 654220RJ

Attach the **side mounting part** on the side of the bed. Place the **floor support** in line with the opening of the side mounting part and place the **upper support** through the side mounting part and inside the floor support so that the longer end of the upper support faces the foot of the bed. Adjust to a suitable height and tighten the thumbscrew on the side mounting part. Make sure that the product is firmly attached.  
-> **The bedside support is ready for use.**

**NOTE!** A safety limit marking (hole) is marked on the lower edge of the upper support (5cm). The upper support must be installed in such a way that the marking is not visible above the floor support.

**NOTE!** THERE MUST NOT BE A CARPET UNDER THE FLOOR SUPPORT

**MUST NOT BE MOUNTED ON MOTORIZED BEDS!**  
In unclear situations, ensure suitability!

## Bedside supports for motorized beds

### Mounting:

Loosen the thumbscrews 1-4 to separate the side mounting part, upper support and floor support from each other.

Place the side mounting part on the side of the bed at the folding point of the motorized lift (or closer to the foot end).

**⚠ The product must not be installed closer to the head end of the bed when viewed from the folding point - DANGER OF CRUSHING!**

Depending on the product, fasten the side mounting part either by tightening the thumbscrews or with a screwdriver using eight screws. Place the floor support in line with the side mounting part and put the upper support in place so that the groove in it is aligned with the pin inside the side mounting part.

**⚠ The upper support can be turned to three different directions. The support should not be installed in such a way that the upper support can be turned above the bed - DANGER OF CRUSHING**

The tube of the upper support itself is also designed in such a way that this is not possible.

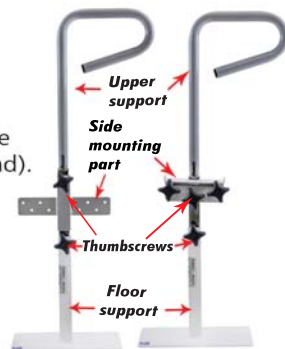
Adjust to a suitable height and tighten the thumbscrew on the side mounting part and the thumbscrew on the floor support.

Make sure that the product is firmly attached  
-> The bedside support is ready for use.

CHECK REGULARLY THE TIGHTNESS OF THE SCREWS.

**⚠ NOTE!**

- \* THE REMOTE CONTROL OF THE MOTORIZED BED MUST NOT BE KEPT IN THE BED!
- \* MUST NOT BE MOUNTED ON HEIGHT-ADJUSTABLE BEDS (so-called hospital and nursing beds)! In unclear situations, ensure suitability!
- \* THERE MUST NOT BE A CARPET UNDER THE FLOOR SUPPORT!

**⚠ NOTE! A safety limit marking (hole) is marked on the lower edge of the upper support (5cm). The upper support must be installed in such a way that the marking is not visible above the floor support.**

## Bedside supports - attached to the bottom

Use eight (8) screws or through-bolts to attach the product, depending on the strength of the bed side. (NOTE! Not included in the product!)

### Mounting - bed on its legs:

Attach one of the side mounting parts to the bottom and/or side of the bed on the opposite side of where the upper support is wished to be used at a distance of about 60-80 cm from the headboard (recommended position; can also be installed according to the user's personal needs) with at least four (4) screws or through-bolts depending on the structure of the bed.

If the bed is so low that the floor support cannot be installed, the side mounting parts must always be attached to both the bottom and the side of the bed with at least eight (8) screws or through-bolts (i.e. a total of at least 16 fastenings).

Place the spacer tube inside the side mounting part already attached to the bed, and after that add the other side mounting part on the other end of the spacer tube. Make sure that the side mounting parts are in a straight line with each other, press the unattached side mounting part tightly against the side of the bed and tighten the two thumbscrews that adjust the width of the spacer tube.

Place the floor support in line with the opening of the side mounting part and place the upper support through the side mounting part and inside the floor support so that the longer end of the upper support faces the foot of the bed.

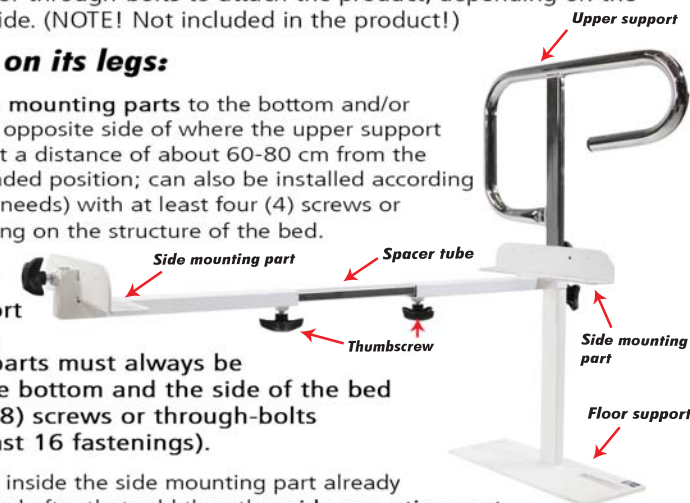
Adjust to a suitable height and tighten the thumbscrew on the side mounting part and the thumbscrew on the floor support. Attach the other side mounting part with at least four (4) screws or through-bolts to the bottom and/or side of the bed. Lastly, make sure that the product is firmly attached  
-> The bedside support is ready for use.

If the upper support is to be installed on both sides of the bed, floor supports must also be used on both sides!

CHECK REGULARLY THE TIGHTNESS OF THE SCREWS.

**⚠ NOTE! THERE MUST NOT BE A CARPET UNDER THE FLOOR SUPPORT!**

**⚠ MUST NOT BE MOUNTED ON MOTORIZED BEDS!**  
In unclear situations, ensure suitability!

**⚠ NOTE!**  
A safety limit marking (hole) is marked on the bottom edge of the upper support (5cm) and on both ends of the spacer tube. The upper support and spacer tube must be installed in such a way that the marking is not visible from inside the floor support and/or side mounting parts. An extra-long upper support model and a longer spacer tube are available as accessories, which increase the height and/or width dimensions.

## Bedside supports - attached to the bottom

Use eight (8) screws or through-bolts to attach the product, depending on the strength of the bed side. (NOTE! Not included in the product!)

### Mounting - bed on its side:

Determine the place of the bedside support suitable for the user (recommendation 60-80 cm from the end of the bed on the head side). Turn the bed on its side.

**!** When turning the bed, always ensure sufficient support or ask an assistant to come along to avoid accidents!

Lift the bed so that you can place one of the side mounting parts in the desired position against the side under the bed with the longer end towards the middle of the bed and fasten with at least 4 screws or through-bolts to the bottom (and/or side) of the bed.

If the bed is so low that the floor support cannot be installed, the side mounting parts must always be attached to both the bottom and the side of the bed with at least eight (8) screws or through-bolts (i.e. a total of at least 16 fastenings).

Place the spacer tube inside the side mounting part already attached to the bed, and after that add the other side mounting part on the other end of the spacer tube. Make sure that the side mounting parts are in a straight line with each other, press the unattached side mounting part tightly against the side of the bed and tighten the two thumbscrews that adjust the width of the spacer tube. Attach the other side mounting part with at least four (4) screws or through-bolts to the bottom and/or side of the bed. Turn the bed right way around.

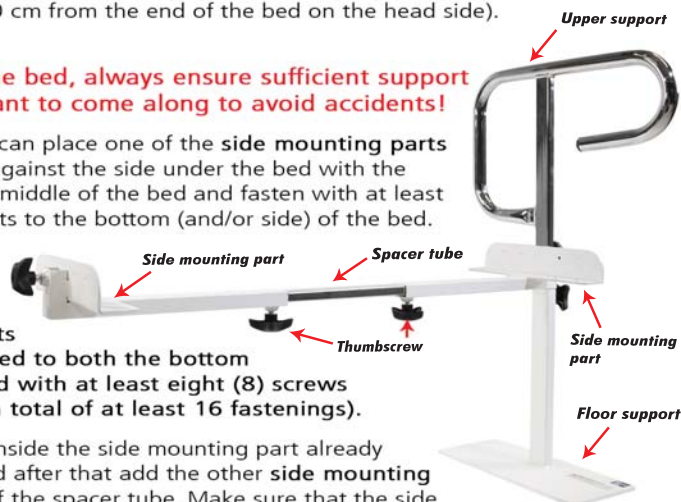
Place the floor support in line with the opening of the side mounting part and place the upper support through the side mounting part and inside the floor support so that the longer end of the upper support faces the foot of the bed. Adjust to a suitable height and tighten the thumbscrew on the side mounting part and the thumbscrew on the floor support. Lastly, make sure that the product is firmly attached -> The bedside support is ready for use.

If the upper support is to be installed on both sides of the bed, floor supports must also be used on both sides!

CHECK REGULARLY THE TIGHTNESS OF THE SCREWS.

**!** THERE MUST NOT BE A CARPET UNDER THE FLOOR SUPPORT!

**!** MUST NOT BE MOUNTED ON MOTORIZED BEDS! In unclear situations, ensure suitability!


**!** **NOTE!**  
A safety limit marking (hole) is marked on the bottom edge of the upper support (5cm) and on both ends of the spacer tube. The upper support and spacer tube must be installed in such a way that the marking is not visible from inside the floor support and/or side mounting parts. An extra-long upper support model and a longer spacer tube are available as accessories, which increase the height and/or width dimensions.

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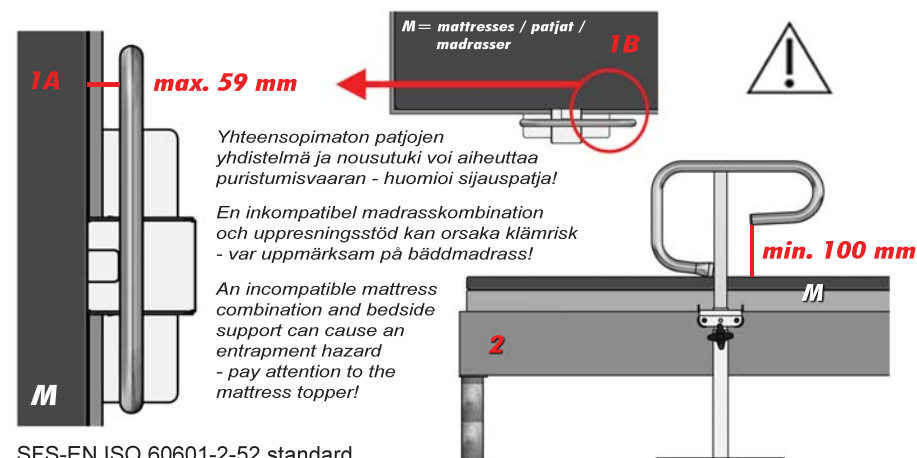**Incident reporting:** Professional users must report to Fimea and the manufacturer incidents regarding situations that have or might have led to a risk to the health of the patient, user, or other person and that result from:

- the characteristics of the product
- malfunction or deterioration in performance
- insufficient labelling
- insufficient or incorrect instructions for use
- another reason related to the use of the product

**Tukimet Oy**  
**Kaivopuistontie 33**  
**26100 RAUMA**  
**FINLAND**  
**[www.tukimet.fi](http://www.tukimet.fi)**

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Copyright Tukimet Oy