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**SIIRI DUSCHSTOL MED HJUL
SIIRI SHOWER CHAIR WITH WHEELS**

345310 | 345312 | 345312V | 345312W

345305 | 345300W | 345305W

345315 | 345316 | 345315W | 345316W

345412W

Copyright Tukimet Oy
versio 2/2025

FI

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bild 7

bild 8

Specifications, general safety instructions & washing

Siiri shower chair with wheels, transport model 345310

Siiri shower chair with wheels, height-adjustable 345312 | 345312V | 345312W

Siiri shower chair XXL with wheels, height-adjustable 345412W

Siiri shower chair with wheels, tilting, height-adjustable 345315 | 345316 | 345315W | 345316W

Siiri shower wheelchair, independent model 345305 | 345300W | 345305W

Fitting of the fabric backrest

Read this manual carefully and follow the assembly instructions. Careful assembly guarantees safe use.

Do not use this equipment without first reading and understanding this instruction manual. If you are unable to understand the warnings, cautions and instructions, contact a healthcare professional, dealer or technical personnel before attempting to install or use this equipment - otherwise, injury or damage may occur. Each individual should always consult with their physician or therapist to determine the best suited assistive aid for them and the proper adjustment and usage of it.

Watch assembly videos on YouTube:

*** Please note that the illustrations in this manual are only for guidance and may contain differences with the actual product.**

Specifications:

Intended use: Siiri shower chair with wheels is a handy help in shower rooms. The wheeled shower chair with hygienic seat is of high quality and meets the needs of both individuals and medical units. The high-quality polyurethane seat is wear-resistant and feels pleasant on the skin. The seat comes with a removable cover. Siiri can be placed in a smaller space thanks to the removable backrest. The armrests can be turned up and the footrests can be moved to the sides.

- Frame white painted aluminum (stainless)
- Seat, armrests and backrest black polyurethane
- Removable backrest and foldable armrests
- Height-adjustable footrests that turn to the sides and are easily removable
- Covered bidet opening
- Bedpan with lid and rails
- Lockable front and rear wheels
- CE-marked medical device

General safety instructions:

- ⚠ - Avoid putting your fingers or other parts of the body in places where they might get pinched
- ⚠ - Operating temperature between -30°C and 40°C (a shower chair that has been in a very hot or cold place should be handled carefully so as not to injure yourself)
- ⚠ - During passenger transport, the bedpan must be kept in place.
- ⚠ If the bedpan is missing, the lid may come off.
- Do not make structural adjustments or changes!
- ⚠ - Regularly check the general condition of the shower chair and the tightness of the fasteners. **A defective device must not be used. Don't try to fix it yourself but take it to a professional!**

Only original spare parts may be used for repair and maintenance in order for the CE approval of the device to remain valid.

Washing: The shower chair should be cleaned regularly with a mild detergent (PH 8-12) and wiped with a damp soft cloth. To remove difficult stains, it is recommended to use grease-dissolving cleaning agents. Alcohol-based disinfectants should be diluted in a ratio of 1:4. The shower chair can be placed under running water at low pressure or sprayed clean. Do not use harsh abrasive materials or bleach to avoid wear and tear and color fading. The seat part and the bedpan can be washed at 85 degrees for a maximum of 1 minute. Can be disinfected mechanically.

- ⚠ **Never use oily substances! The surface of the product becomes slippery and the grip does not hold!**

Siiri shower chair with wheels, transport model

- Front and rear wheels 125 x 30 mm
- Total width: 57 cm
- Total height: 100 cm
- Bidet opening size: 19.5 cm x 26.5 cm
- Seat size: 43 x 45 cm
- Seat height: 53 cm
- Seat width: 46 cm
- Weight: 13 kg
- User's max. weight: 180 kg

Commissioning:

Remove all parts of the product from the packaging.

fig. 1

Insert backrest into the tubes at the back of the frame, select appropriate height and tighten with the handwheels (fig. 1).

Installing the footrests: the top bracket of the footrest is pressed down into the frame tube so that the footboard is sideways to the frame (fig. 2).

fig. 2

Next, turn the footrest towards the center until it locks into place and makes a "click" sound (fig. 3).

fig. 3

Repeat with the other footrest.

The height of the footboards can be adjusted without tools by using the handwheels (fig. 4).

fig. 4

The footrests can be released and removed by pressing on the lever (fig. 5).

fig. 5

Siiri shower chair with wheels, height-adjustable

345312 | 345312V

- Front and rear wheels 125 x 30 mm
- Total width: 57 cm
- Total height: 99 cm - 105 cm
- Bidet opening size: 19.5 cm x 26.5 cm
- Seat size: 43 x 45 cm
- Seat height: 49 - 55 cm
- Seat width: 46 cm
- Weight: 13 kg
- User's max. weight: 150 kg

345312W

- Front and rear wheels 125 x 30 mm
- Total width: 66 cm
- Total height: 97 cm - 103 cm
- Bidet opening size: 19.5 cm x 26.5 cm
- Seat size: 56 x 42 cm
- Seat height: 49 - 54 cm
- Seat width: 57 cm
- Weight: 17 kg
- Max. user weight: 200 kg

Commissioning:

Remove all parts of the product from the packaging. Install the wheels into the legs of the chair, adjust to suitable height and lock with the quick-lock clip (fig. 1).

fig. 1

The chair has three height options (fig. 2).

fig. 2

NOTE! The top hole of the height adjustment tube must not be installed in the bottom hole of the frame (fig. 3).

For a higher seat height, a longer height adjustment tube is available as an accessory (see page 8).

fig. 3

Make sure that the clips are properly attached!

Alternatively, the package also includes mounting bolts for a more permanent installation of the wheels (fig. 4).

Insert backrest into the tubes at the back of the frame, select appropriate height and tighten with the handwheels (fig. 5).

fig. 4

Install the footrests (see pictures on the previous page): the top bracket of the footrest is pressed down into the frame tube so that the footboard is sideways to the frame. Turn the footrest towards the center until it locks into place and makes a "click" sound. Repeat with the other footrest. The height of the footboards can be adjusted without tools. The footrests can be released and removed by pressing on the lever.

fig. 5

Siiri shower chair XXL with wheels, height-adjustable

- Front and rear wheels 125 x 65 mm
- Double brake function that locks both the wheel and the swivel bearing
- Overall width: 71 cm
- Overall height: 95 cm - 100 cm
- Bidet opening size: 19 cm x 41 cm
- Seat size: 56 x 50 cm
- Seat height: 48 - 53 cm
- Seat width: 61.5 cm
- Weight: 19 kg
- Max. user weight: 270 kg

Assembly:

See the product "Siiri shower chair with wheels, height-adjustable" instruction page for instructions on installing the wheels, backrest and footrests.

fig. 1

NOTE! The top hole of the height adjustment tube must not be installed in the bottom hole of the frame (fig. 1). For a higher seat height, a longer height adjustment tube is available as an accessory.

The armrest is locked in the down position. To release and raise the armrest, pull the red knob (fig. 2).

fig. 2

When you want to use the armrest again, press the armrest down (fig. 3).

Make sure the armrest locks!

fig. 4

All four wheels are lockable with a double brake function. When the blue lever is pressed down, the wheel is locked. It is released by lifting the lever back up (fig. 4). **When getting off the chair and sitting in it, all four wheels should always be locked!**

fig. 3

Siiri shower chair with wheels, tilting, height-adjustable

345315 | 345316

- Front and rear wheels 125 x 30 mm
- Total width: 57 cm
- Total height: 100 cm - 106 cm
- Tilt angles: front 50° / back 30°
- Bidet opening size: 19 cm x 36 cm
- Seat size: 43 x 45 cm
- Seat height: 55 cm - 59 cm
- Seat width: 46 cm
- Weight: 14 kg
- User's max. weight: 130 kg

345315W | 345316W

- Front and rear wheels 125 x 30 mm
- Total width: 66 cm
- Total height: 100 cm - 106 cm
- Tilt angles: front 50° / back 30°
- Bidet opening size: 19 cm x 36 cm
- Seat size: 56 x 51 cm
- Seat height: 55 cm - 60 cm
- Seat width: 57 cm
- Weight: 19,5 kg
- User's max. weight: 200 kg

Commissioning:

See the product "Siiri shower chair with wheels, height-adjustable" instruction page for instructions on the wheels, backrest and footrests.

Insert the headrest into the black bracket in the middle of the push bar, set it to a suitable height and tighten with the handwheel (fig. 1 & 2).

The upper handwheel adjusts the distance of the headrest (fig. 3).

Tilting:

The tilt angle can be adjusted by using the handle found on the push bar (fig. 4).

Squeeze the handle: the hold releases and the tilt of the chair can be adjusted. Let go off the handle when the chair is in the desired position. The position locks.

⚠ If the user sits in the chair alone for a longer period of time, the wheels should be locked so that they point away from the product and thus give better support.

fig. 1

fig. 2

fig. 3

fig. 4

Siiri shower wheelchair, independent model

345305, height-adjustable

- Rear wheels 24" polyurethane
- Front wheels 125 x 30 mm
- Total width: 65 cm
- Total height: 97 - 102 cm
- Bidet opening size: 19 cm x 36 cm
- Seat size: 43 x 45 cm
- Seat height: 49 - 55 cm
- Seat width: 46 cm
- Weight: 17 kg
- Max. user weight 130 kg

345300W | 345305W, height adj.

- Rear wheels 24" polyurethane
- Front wheels 125 x 30 mm
- Total width: 74 cm
- Total height: 100 cm | 97-102.5 cm
- Bidet opening size: 19 cm x 36 cm
- Seat size: 56 x 42 cm
- Seat height: 53 cm | 49-55 cm
- Seat width: 57 cm
- Weight: 18 kg
- Max. user weight: 200 kg

Commissioning:

Remove all parts of the product from the packaging. Insert backrest into the tubes at the back of the frame, select appropriate height and tighten with the handwheels (fig. 1).

Install the footrests (see detailed instructions on the instruction page of the product "Siiri shower chair with wheels, transport model").

Rear wheel brakes:

To lock the rear wheels, press the brake lever down until it clicks (fig. 2).

Height-adjustment of model 345305:

The height of the front wheel is adjusted by removing the quick release clip, selecting the desired height and attaching the quick release back in place (fig. 3 / page 22).

⚠ Make sure that the clips are properly attached!

Height adjustment of the rear wheels is done by changing the position of the axle (fig. 4-6).

fig. 1

fig. 2

fig. 3

fig. 4

fig. 5

fig. 6

Fitting of the fabric backrest

The 345202 fabric mesh backrest available as an accessory can be installed on all standard Siiri shower chairs (models with a seat size of 43 x 45 cm) and 345202W on models 345312W and 345315W.

Start by removing the PU padding part from the backrest frame by unscrewing the 4 cross-headed screws.

Then place the mesh backrest on the frame of the backrest so that the buckles are on the backside of the chair and the lowest buckle strip goes under the armrests and the middle strip over them (the fabric backrest has recesses for the armrests).

Thread the other end of each of the three buckle strips through the buckle. Tighten to the desired position and fasten by pressing the Velcro straps together (fig. 1).

On the upper edge of the mesh backrest, there are two thinner fastening strips, which prevent the mesh backrest from sliding downwards.

With standard transport models and the independent Siiri, the straps are pulled crosswise and attached to the Velcro between the middle buckle strap (fig. 2).

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Place both straps hanging over the top of the push bar (fig. 3). Pull the left strap from under the headrest bracket (fig. 4). Place the Velcro loop area over the hook area and fasten together (fig. 5).

Pull the right strap from under the headrest bracket and over the left strap (fig. 6). Place the Velcro loop area over the hook area and fasten together (fig. 7).

When installed in this way, the backrest remains sufficiently taut and does not fall down or press against the handwheel of the headrest (fig. 8).

fig 1

fig 2

fig 3

fig 4

fig 5

fig 6

fig 7

fig 8

Accessories:

SUPPORT FOR AN AMPUTATED LEG

Product no.: 345 220 left 345 221 right

SAFETY BAR

Product no.: 345 210 / 345 210W

SOLE STRAP

Product no.: 345 223

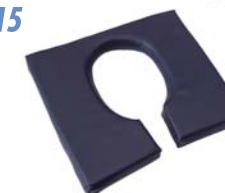
SAFETY BELT

Product no.: 345 224

SEAT, FRONT EDGE CLOSED

Product no.: 345 218 / W / HD-12 / HD-15
incl. seat and cover

SEAT CUSHION WITH OPENING

Product no.: 345 227

LONGER ADJUSTMENT PIPE FOR WHEEL (+ 50 mm)

Product number: 345 145

SIDE SUPPORT

Product no.: 345 228

FALLING PROTECTION, a pair

Product no.: 345 208

TILTING ASSISTANCE for Siiri independent shower wheelchair

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Incident reporting: Professional users must report to Fimea and the manufacturer incidents regarding situations that have or might have led to a risk to the health of the patient, user, or other person and that result from:

- the characteristics of the product
- malfunction or deterioration in performance
- insufficient labelling
- insufficient or incorrect instructions for use
- another reason related to the use of the product

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