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EN

* Store the manual for future reference *

Specifications

Assembly

Maintenance and washing instructions

Parts list

***Read this manual carefully and follow the assembly instructions.
Careful assembly guarantees safe use.***

Do not use this equipment without first reading and understanding this instruction manual. If you are unable to understand the warnings, cautions and instructions, contact a healthcare professional, dealer or technical personnel before attempting to install or use this equipment - otherwise, injury or damage may occur. Each individual should always consult with their physician or therapist to determine the best suited assistive aid for them and the proper adjustment and usage of it.

* Please note that the illustrations in this manual are only for guidance and may contain differences with the actual product.

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Se installationsfilmer på YouTube:

General safety instructions:

- Avoid putting your fingers or other parts of the body in places where they might get pinched
 - Operating temperature between -30°C and 40°C (product that has been in a very hot or cold place should be handled carefully so as not to injure yourself)
 - Do not make structural adjustments or changes!
 - Regularly check the general condition of the product and the tightness of the fasteners. **A defective device must not be used.**
- Don't try to fix it yourself but take it to a professional!

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Watch installation videos on YouTube:

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Specifications:

Intended use: The shower chair is an everyday aid that greatly facilitates the independent or assisted washing of people with reduced mobility. Shower chairs are a familiar sight in swimming halls, hospitals and nursing homes, but they are also a good aid for making showering at home easier. The seat is made of water-resistant material. The removable plastic seat is easy to keep clean. A shower chair together with the accessory backrest offers support for maintaining the position and brings relaxation to the bath time. The armrests and adjustable legs make it easier to stand up.

- Height adjustment with clips at 2.5 cm intervals
- CE-marked medical device

**Anneli shower chair
with enclosed seat
Product no.: 337600**

**Anneli EXW extra-wide
with enclosed seat
Product no.: 337600W**

**Anni shower chair
with hygienic cutout seat
Product no.: 337610**

**Anni EXW extra-wide
with hygienic cutout seat
Product no.: 337610W**

- White aluminum frame with armrest
- Leveling foot
- Detachable, impact resistant plastic seat either with or without a hygienic cutout
- Available in two different widths; regular or extra-wide
- Backrest is sold separately as an accessory item (product no.: 337220)
- Max. user weight: 180 kg

**Auli XL shower chair
with hygienic cutout seat
Product no.: 337630L**

**Aulikki XL shower chair
with enclosed seat
Product no.: 337640L**

- Wide frame with armrest
- Leveling foot
- Detachable, impact resistant plastic seat either with or without a hygienic cutout
- Max. user weight: 300 kg

Salli shower chair Product no.: 337170
- Detachable, impact resistant enclosed plastic seat
- Max. user weight: 120 kg

Annu shower stool Product no.: 337622
- Detachable, impact resistant enclosed plastic seat
- Max. user weight: 150 kg

Assembly:

Seat:

Place the seat part on the chair frame. Press the seat firmly. Check by lifting the chair from the seat that it is securely attached to the frame. The front edge of the shower chair is in the foreground in fig 1.

NOTE!

* Avoid sitting on the very edge of the seat so that the seat does not detach from the frame!

fig. 1

Height adjustment:

Remove the blue clips from the frame by pulling them towards you. Push the height adjustment legs into the frame. Choose the appropriate height and lock the height adjustment legs to the frame by pushing the clip back into place (fig. 2).

* Make sure that the clips are properly attached and that they completely penetrate both tubes.

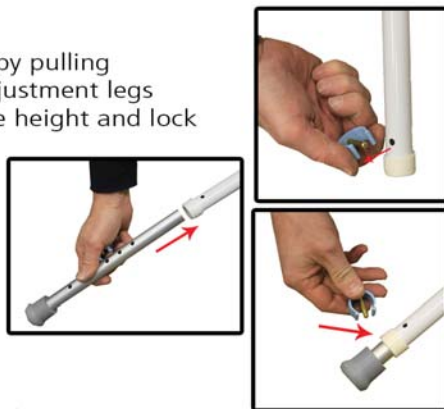
fig. 2

Tilt adjustment:

One of the legs of the product is the levelling adjustment leg, which does not have a grey rubber stopper at all. By adjusting the swivel screw (fig. 3) on it, the possible tilt of the bathroom floor can be levelled.

fig. 3

Back rest (separate accessory):

Insert the backrest tubes into the frame from the side of the straight intermediate tube (= backside of the chair).

There are two different depths to choose from for the backrest. Choose the one that suits you best and lock the backrest to the frame with the clips (fig. 4).

* Make sure that the clips are properly attached and that they completely penetrate both tubes.


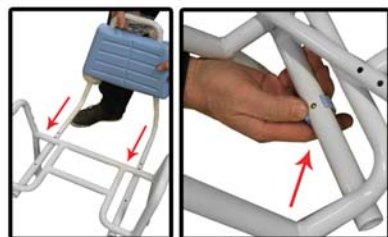
fig. 4

SHOWER CHAIR SALLI FOLDABLE 337170

Assembly instructions:

1. Fold the leg at the back of the chair frame sideways and fold the leg at the front perpendicularly away from the frame.
2. Lift the seat frame and align it with the mounting holes in the legs. Make sure both edges of the seat frame are aligned and press down so that the ends of the tubes on both sides go all the way to the bottom of the holes.

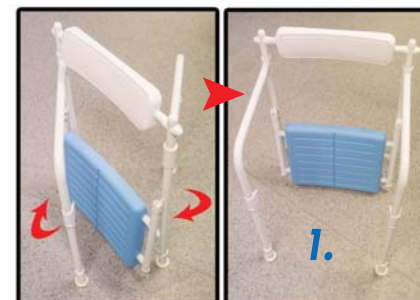3. If the seat is not in place, place the seat on the chair frame and press the seat firmly into place. Check by lifting the chair by the seat to make sure it is securely attached to the frame.

4. Remove the blue clips from the frame by pulling them towards you.

5. Choose the appropriate height and lock the height adjustment legs to the frame by pushing the clip back into place. Make sure that the clips are properly attached and that they completely penetrate both tubes.

Maintenance:

To ensure functionality and safety, take the following measures.

Check the general condition of the device at regular intervals:

- The general condition of the parts (seat, frame, legs and, if applicable, backrest): cracked or damaged ones should be replaced with new ones.
- The condition of the rubber stoppers on the legs; replace a worn or damaged one with a new one.
- Leg and backrest clips: check condition and replace if damaged.
- Check that all parts are properly fastened: adjust if necessary.

If something out of the ordinary is observed during use, the device should be serviced immediately.

CAUTION: Only original Tyke® spare parts may be used for repair and maintenance in order for the CE approval of the device to remain valid.

Washing:

Hold the shower chair clean of all dirt by wiping with a damp cloth and a mild disinfectant cleaner / detergent so that it remains functional.

If necessary, the plastic seat and the plastic part of the backrest can be washed in an autoclave at 85 degrees for a maximum of 1 minute.

The shower chair can also be placed under running water at low pressure or sprayed clean with the shower head. Do not use harsh abrasive materials or bleach to avoid wear and tear and color fading. The product can be disinfected.

Never use oily substances! The surface will become slippery and your grip won't hold!

To replace the consumables listed here, you can ask our customer service for the more detailed service instructions: tukimet@tukimet.fi

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Incident reporting: Professional users must report to Fimea and the manufacturer incidents regarding situations that have or might have led to a risk to the health of the patient, user, or other person and that result from:

- the characteristics of the product
- malfunction or deterioration in performance
- insufficient labelling
- insufficient or incorrect instructions for use
- another reason related to the use of the product

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Copyright Tukimet Oy
Tukimet Oy