Tekniset tiedot:

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Maintenance:

To ensure functionality and safety, take the following measures.

Check the general condition of the device at regular intervals:

- The general condition of the parts: cracked or damaged ones should be replaced with new ones.
- Fasteners and/or clips: check condition and replace if damaged.
- Check that all parts are properly fastened: adjust if necessary.

If something out of the ordinary is observed during use, the device should be serviced immediately.

CAUTION: Only original Tyke® spare parts may be used for repair and maintenance in order for the CE approval of the device to remain valid.

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Se installationsfilmer på YouTube: 

General safety instructions:

- Avoid putting your fingers or other parts of the body in places where they might get pinched
- Operating temperature between -30°C and 40°C (product that has been in a very hot or cold place should be handled carefully so as not to injure yourself)
- Do not make structural adjustments or changes!
- Regularly check the general condition of the product and the tightness of the fasteners. A defective device must not be used. Don't try to fix it yourself but take it to a professional!

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Watch installation videos on YouTube: 