USER MANUAL | BEDIENUNGSANLEITUNG

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500100MU
WHEELLATOR® Black / Schwarz / Musta / Svart

500100PU
WHEELLATOR® Red / Rot / Punainen / Röd

Copyright Tukimet Oy
ver. 1/2025

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EN

* Save the user manual for future use *

Specifications & important measures	page 3
Labelling information	page 4
Commissioning	page 4-6
- Rear wheels	page 4
- Handlebars / Footboards	page 5
- Footboard & back support height adjustment	page 6
Folding the Wheelllator up	page 6
Parking and operating brakes	page 7
Turning the Wheelllator into a handrim-propelled model	page 8
Part list & recycling	page 9-10
Maintenance and cleaning instructions	page 11
Warranty, liability & limitations	page 12
MDR incident reporting (EU) Symbols Check-list & serial No.	page 44-46

* Please note that the illustrations in this manual are only for guidance and may contain differences with the actual product.

The manual can also be
downloaded electronically at:
www.tukimet.fi/ohjekirjat

Watch the unboxing
video in YouTube:

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 **WHEELLATOR®**

NOTE: Check all parts for shipping damage. In case of shipping damage, DO NOT use. Contact importer/reseller for further information.

Do not use this equipment without first reading and understanding this instruction sheet. If you are unable to understand the Warnings, Cautions and Instructions, contact a healthcare professional, dealer or technical personnel before attempting to install or use this equipment - otherwise, injury or damage may occur. Each individual should always consult with their physician or therapist to determine the best suited assistive aid for them and the proper adjustment and usage of it.

Specifications

Intended purpose: For people who can use a rollator. You need to be able to walk by yourself. It is not made for wheelchair users. The Wheelator supports users with limited mobility e.g. when shopping or taking a walk outdoors. The seat allows a place to rest during longer distances. The device must be used on level and solid ground only.

- For indoor & outdoor use
- Frame made of powder coated steel piping
- Handlebars made from chrome plated steel piping
- Height adjustable handlebars with height markings that make adjustments easier
- Grips made of soft plastic for comfortable use (incl. small amounts of PVC rubber that may cause allergies. OM032U handles, suitable for rubber allergy, are available as accessories)
- Cable brakes act as both parking and service brakes
- Seat made from molded polyurethane
- Front wheels: size 200 x 50 mm, puncture-resistant polyurethane, nylon rims with ball bearings. Front forks with double ball bearings.
- Standard version has 24" solid rubber rear wheels (inflatable wheels are sold separately)
- Max. weight of the load in the basket: 9 kg. Items placed in the basket should not protrude from the basket.
- Operating temperature between -30°C and 40°C (Handle a device that has been in a very hot or cold place with care so that you do not injure yourself).
- Expected lifespan: 10 years
- CE marked medical device

The Wheelator has been patented and the brand & model are protected.

Conforms to the Requirements of Medical Device Regulation (EU) 2017/745 relating to Medical Devices Class 1 Product. Meets, where applicable, requirements and test methods below:

- ISO 11199-2, Rollators, Part 2: The requirements of Assistive products for walking manipulated by both arms
- ISO 7176-8, Wheelchairs, Part 8: Requirements and test methods for static, impact and fatigue strengths

Important measures:

Handle height adjustment range: 79-91 cm
Overall dimensions: 680 x 850 x 790 mm
Folded dimensions: 680 x 980 x 610 mm (without back wheels 420 mm)
Max width: 68 cm
Seat height: 53 cm
Seat width (between handles): 43 cm
User max. weight: 150 kg
Weight: 16 kg

 **WHEELLATOR®**

Labelling information:

The product info label can be found under the seat in Finnish, English and German.


Commissioning:

Remove the Wheelllator from its package by opening the box from its topside. Carefully lift all parts out of the box.

Rear wheels:

Insert the quick-lock axle through the rear wheel on the rim side while pushing down the button on the quick-lock and insert the combination into the brake plate (figure 1).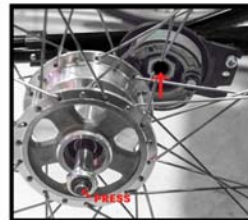

The rear wheels are removed by pressing in the middle and pulling the wheel simultaneously outward (figure 2).

CAUTION! When fitting the rear wheels, always make sure that the axle has locked in properly! Adjust if necessary.

Handlebars:

Remove the black plugs protecting the frame tubes (figure 1). The hand screws have been turned inward to keep them safe from shipping damages. Unattach the screws and turn them around, so that the black plastic hand piece is outside and the metal bolt inside the device (figure 2). Insert the handlebars into the frame pipes, make sure that the bars go on the same side where the brake cable is attached to the drum brake. Make sure that the brake cable always runs on the outer side of the handle. Lock the handlebars at a suitable height by tightening the hand screws (figure 3).

CAUTION:

- ⚠ Max. allowed height: the third point **MUST NOT** be visible above the end of the frame pipe!
- ⚠ The hand screws **MUST** be tight enough to ensure that handle position does not change during use!
- ⚠ After adjusting handle height **ALWAYS** check the brakes!
- ⚠ The maximum allowed lateral adjustment angle between the longitudinal axis of the handle and the direction of movement is 30° (figure 4 / when walking forward).

Footboards:

Insert the footboards by their chromed pipes inside the slightly bigger diameter pipes next to the rear wheels. Make sure that the footboards are on correct sides; both should fold open towards one another. Select footboard distance by pushing the footboard to the desired position inside the pipe. There are three distance options. After selecting the distance, press on the quick-release lock (figure 5).

- ⚠ **CAUTION! Ensure that the c-clip is properly attached and passes through both pipes!**

Footboard height adjustment:

Footboard height can be adjusted by loosening the nut with a hex key (figure 1). There are three height options.

CAUTION! After adjusting the height, you must ensure that the footboard is still properly attached!

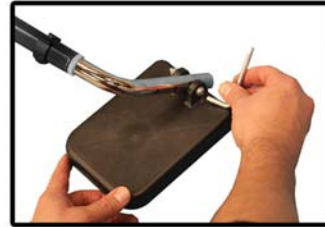
figure 1

Back support height adjustment:

Release the back support by loosening the back support screws with a hex key. Adjust to the desired height and tighten the screws (figure 2).

CAUTION! After adjusting the height, you should always check that the back support is securely attached and that all screws (4 pcs) are tightened.

figure 2

Folding up:

Lift the locking spring under the seat and tilt the Wheelllator forward. The rear pipe is released, and you can store and transport your Wheelllator in a smaller space (figure 3).

CAUTION! Mind your hands when folding. Make sure that they do not get caught between the locking spring and rear pipe or between the rear and front pipes!

figure 3

Parking brake:

Press both brake handles down (figure 1). The handles snap into the 'locked' position.

CAUTION! When opening the parking brakes, be careful not to trap any body parts in between.

CAUTION! The brakes **MUST** be in the locked position before sitting down or standing up!

figure 1

Operating brake:

Pull upwards to use the brakes.

Lubricate the brake handles' moving parts regularly (figure 2). Use only Würth saBesto multi-purpose grease IV or other similar synthetic high-performance grease, that meets the requirements of the NSF H1 specification.

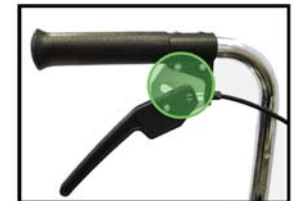
figure 2

ALWAYS before use, check that the brakes work efficiently and if necessary, use the adjustment screw (figure 3).

figure 3

CAUTION!

CAUTION! Never use the Wheelllator as a kick scooter (figure 4)!

CAUTION! The footboards should be turned to the sides when getting up from the Wheelllator. Never stand up while on the footboards. If you step on the footboard, the Wheelllator may tip over (figure 5).

CAUTION! **DO NOT** stand on the device!

figure 4

figure 5

Turning the Wheellator into a handrim-propelled model:

Loosen the hand screws on the side of the handlebars. Drop both handlebars downwards so that they do not get in the way when using the handrims. You can also rotate both handlebars towards your stomach to make a "seat belt". When an assistant is pushing the Wheellator, the handlebars should be turned backwards.

Tighten the hand screws once handlebars are set to the desired position.

Turn the footboards down and check that they are securely attached.

⚠ When being pushed on the Wheellator by an assistant, **ALWAYS** keep your feet on the footboards.

CAUTION!

- ⚠ The Wheellator is not meant to replace a wheelchair, nor to be propelled by the handrims over long distances.
- ⚠ When moving forward using the handrims, ensure your hands are on the handrim and that your fingers are not between the wheels' spokes. (figure 1)

Be aware of changes to weight distribution that may cause the Wheellator to tip:

- ⚠ **DO NOT** attempt to reach objects on the floor while seated.
- ⚠ When sitting and moving by propelling the handrims, **DO NOT** lean suddenly and swiftly forward.

figure 1

To replace the consumables listed here, you can ask our customer service for the more detailed service instructions: tukimet@tukimet.fi

NOTE! Service and replacement of parts should only be performed by a qualified professional!

Part list:

Part list:

Name	Pcs	Product nro.	Material
1 Front frame	1	PV500100-1	M
2 Back frame	1	PV500100-2	M
3 Complete handle bar	1	V500100-5	M/P/C
4 Folding bearing and supplies	2	339600-91	M
5 Hand screw and supplies	2	339600-92	C/M
6 Locking spring and supplies	1	339600-93	M
7 Seat and screws	1	500100-100	C/M
8 Front wheel and supplies	2	339600-94	C
9 Plastic fork and supplies	2	339600-95	C
10 Back wheel, grey PU with handrim	2	OM0114	C
11 Brake plate and supplies	2	500100-96	M
12 Quick release axle	2	OM0112	M
13 Brake cable	2	OM013S	C
14 Grip (handle bar cover)	2	OM033	P
15 Brake complete	2	339905S	C
16 Footboard complete	2	500100-97	P/M
17 Back support	1	500100-90	C
18 Back support fixtures and supplies	2	OM0251	M

Recycling:

When recycling, the product's frame is separated from all the other parts. After unscrewing and dismantling the screws and other connections, the parts should be sorted according to material: metal (M), plastic (P) or combined materials (C). Parts marked here as combined materials are layered so that separating them to metal or plastic parts is not possible.

Note!

Different authorities/recycling companies handle recycled materials in different ways. When in any doubt over the best way to proceed when recycling, contact the relevant governing body.

Maintenance and cleaning instructions:

Regular maintenance will increase the life and safety of your Wheellator.

-  **CAUTION! Check the operation of the brakes ALWAYS before use!**
-  **Before using, make sure the device is locked into the open position.**

Get your Wheellator checked by an assistive aid professional **once a year**. The checkup must include inspection of the tightness of all screws, nuts, and bolts, and that the brakes are in good condition and function properly. If necessary, the brakes should be adjusted by using the adjustment screw. The moving parts of the brake lever should also be lubricated (see p. 7). The rubber handles must be checked and changed **at least every 2 years**.

Only original Wheellator® spare parts may be used for repair and maintenance.

-  **Do not make any adjustments or modifications that may affect the device's structural safety. Damaged or broken device should not be used.**
-  **Do not use the product on stairs or escalators.**
-  **Take special care when moving with the product on steep hills or uneven terrain or when crossing over thresholds or curb edges.**
-  **When lifting the product, for example, to a car, use the rear pipe. Be careful and apply ergonomic lifting techniques.**

When assigning the device to a new user, the user manual must be included.

Clean your Wheellator regularly. Use a mild detergent and water to wash the device. For difficult stains, we recommend grease-dissolving detergents with a pH between 5 and 9 or a 70% disinfectant. Washing temperature max. 60°C. Do not use rubbing alcohol-based cleaners or strong solvents.

-  **CAUTION: Make sure no water remains inside the frame pipe!**

Warranty and liability:

Tukimet Oy warrants the frame against defects in material and workmanship for 12 months after the purchase of the original user. Warranty is based on statutory provisions.

Tukimet Oy's liability shall be limited to the greatest extent allowed by law. Any damage caused by abuse or improper use, adjustment or maintenance as well as normal wear and tear is not covered by warranty. In no event shall Tukimet Oy or its agents be liable for incidental or consequential damages, whether the claim is based upon contract, warranty, negligence or product liability, including without limitation, loss to property other than the product itself, loss of use of any property, or other economic losses. Neither Tukimet Oy nor any distributor or dealer shall be liable for contribution or indemnification, whatever the cause.

FURTHER LIMITATIONS:

1. Tukimet Oy does not warrant tires and tubes, and accessories or any other aftermarket parts.
2. Tukimet Oy does not warrant against any damage from neglect, misuse or damage incidental to any automobile or conveyance incident or accident.
3. Tukimet Oy does not warrant against any damage resulting from exceeding the weight limit of 150 kg.
4. Tukimet Oy does not warrant a device from which the original serial number tag has been removed.
5. Tukimet Oy does not warrant defects caused by not following the user manual.
6. Tukimet Oy does not warrant damages caused by inadequate assembly, handling or strain.
7. Tukimet Oy does not warrant damages caused by inadequate maintenance and care.
8. Tukimet Oy does not warrant damages caused by changes to the product.
9. Tukimet Oy does not warrant damages caused by inadequate fitting
10. Tukimet Oy does not warrant damages caused by crashes, collisions or other accidents.
11. Tukimet Oy does not warrant damages caused by force majeure.
12. Tukimet Oy does not warrant damages due to wear and tear of parts.
13. Tukimet Oy does not warrant insignificant defects or divergences that are irrelevant for the value and use of the product.
14. Tukimet Oy does not warrant personal and other property damages.
15. Tukimet Oy does not warrant damages due to other circumstances for which it cannot be held responsible.

European legislation for medical devices (MDR)

Incident reporting: Any serious incident caused by a medical device should be reported to the local authority (applicable within the European Union). More detailed information about reporting in Finland can be found on page 44.

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Incident reporting in Finland: Professional users must report to Fimea and the manufacturer incidents regarding situations that have or might have led to a risk to the health of the patient, user, or other person and that result from:

- the characteristics of the product
- malfunction or deterioration in performance
- insufficient labelling
- insufficient or incorrect instructions for use
- another reason related to the use of the product

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