

### Mounting:


Loosen the hex screws 1-3 and thumbscrew 4 to separate the side mounting part, upper support and floor support from each other.

Place the side mounting part on the side of the bed at a distance of about 60-80 cm from the headboard (recommended position; can also be installed according to the user's personal needs).

The side part can be placed not only on the top edge of the side of the bed, but also on the bottom edge when turned the other way around, for example in the case of a frame mattress bed.

Fasten the side mounting part by tightening hex screws 1 and 2. Place the floor support in line with the opening of the side mounting part and place the upper support through the side mounting part and inside the floor support so that the longer end of the upper support faces the foot of the bed.

Adjust to a suitable height and tighten hex screw 3 on the side mounting part and thumbscrew 4 on the floor support. Lastly, make sure that the product is firmly attached.

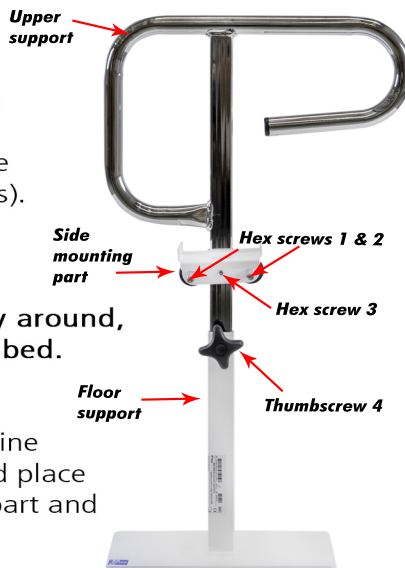

**NOTE!** A safety limit marking (hole) is marked on the lower edge of the upper support (5cm). The upper support must be installed in such a way that the marking is not visible above the floor support.



**MUST NOT BE MOUNTED ON MOTORIZED BEDS!** In unclear situations, ensure suitability!



**THERE MUST NOT BE A CARPET UNDER THE FLOOR SUPPORT!**

The product installer must read the installation manual and watch the installation video. The installation does not require special skills and can be performed by the user.

More comprehensive user manuals can be found here:  
[www.tukimet.fi/ohjekirjat](http://www.tukimet.fi/ohjekirjat)

**Watch the commissioning video on YouTube:**



Check regularly the general condition of the product and the tightness of the screws. Keep the product clean of all dirt by wiping it with a damp cloth and a mild detergent to keep it in good shape. Have a faulty product serviced at your own assistive device services facility!