


Use eight (8) screws or through-bolts to attach the product, depending on the strength of the bed side. (NOTE! Not included in the product!)

Mounting: **654100R | 654120R**

Loosen the thumbscrew to separate the side mounting part from the upper support. Place the side mounting part on the side of the bed at a distance of about 60-80 cm from the headboard (recommended position; can also be installed according to the user's personal needs). Attach the side mounting part with 8 screws or through-bolts depending on the strength of the side of the bed.

Place the upper support through the side mounting part so that the longer end of the upper support faces the foot of the bed. Adjust to a suitable height and tighten the thumbscrew on the side mounting part. Lastly, make sure that the product is firmly attached.

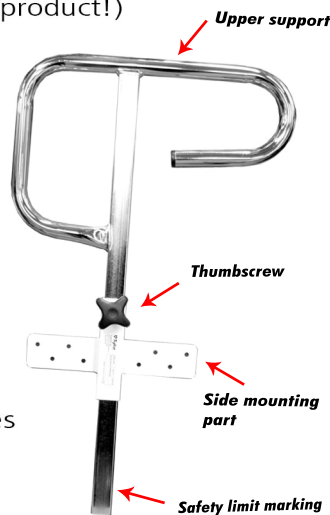

ATTACHMENT STRENGTH DEPENDS ON THE STRUCTURE OF THE WOODEN PARTS OF THE BED FRAME. IF POSSIBLE, IT IS RECOMMENDED TO ALWAYS USE A FLOOR SUPPORT. The screw mounted bedside support is also available in combination with the floor support or the floor support can be purchased as an accessory.

654100RJ | 654120RJ | 654200RJ | 654220RJ

Attach the side mounting part on the side of the bed. Place the floor support in line with the opening of the side mounting part and place the upper support through the side mounting part and inside the floor support so that the longer end of the upper support faces the foot of the bed. Adjust to a suitable height and tighten the thumbscrew on the side mounting part. Make sure that the product is firmly attached.

NOTE! A safety limit marking (hole) is marked on the lower edge of the upper support (5cm). The upper support must be installed in such a way that the marking is not visible above the floor support.



MUST NOT BE MOUNTED ON MOTORIZED BEDS!
In unclear situations, ensure suitability!



THERE MUST NOT BE A CARPET UNDER THE FLOOR SUPPORT!

The product installer must read the installation manual and watch the installation video. The installation does not require special skills and can be performed by the user.

More comprehensive user manuals can be found here:
www.tukimet.fi/ohjekirjat

Watch the commissioning video on YouTube:



Check regularly the general condition of the product and the tightness of the screws. Keep the product clean of all dirt by wiping it with a damp cloth and a mild detergent to keep it in good shape. Have a faulty product serviced at your own assistive device services facility!