



## Mounting:

Loosen the thumbscrews to separate the side mounting part, upper support and floor support from each other. Place the side mounting part on the side of the bed at the folding point of the motorized lift (or closer to the foot end).

 **The product must not be installed closer to the head end of the bed when viewed from the folding point - DANGER OF CRUSHING!**

Depending on the product, fasten the side mounting part either by tightening the thumbscrews or with a screwdriver using eight screws. Place the floor support in line with the side mounting part and put the upper support in place so that the groove in it is aligned with the pin inside the side mounting part.

 **The upper support can be turned to three different directions. The support should not be installed in such a way that the upper support can be turned above the bed - DANGER OF CRUSHING!**

The tube of the upper support itself is also designed in such a way that this is not possible.

Adjust to a suitable height and tighten the thumbscrew on the side mounting part and the thumbscrew on the floor support. Make sure that the product is firmly attached.

 **NOTE!**

- \* THERE MUST NOT BE A CARPET UNDER THE FLOOR SUPPORT!
- \* THE REMOTE CONTROL OF THE MOTORIZED BED MUST NOT BE KEPT IN THE BED!
- \* MUST NOT BE MOUNTED ON HEIGHT-ADJUSTABLE BEDS (so-called hospital and nursing beds)! In unclear situations, ensure suitability!

The product installer must read the installation manual and watch the installation video. The installation does not require special skills and can be performed by the user.


More comprehensive user manuals can be found here:  
[www.tukimet.fi/ohjekirjat](http://www.tukimet.fi/ohjekirjat)

**Watch the commissioning video on YouTube:**



Check regularly the general condition of the product and the tightness of the handscrews. Keep the product clean of all dirt by wiping it with a damp cloth and a mild detergent to keep it in good shape. Have a faulty product serviced at your own assistive device services facility!



 **NOTE! A safety limit marking (hole) is marked on the lower edge of the upper support (5cm). The upper support must be installed in such a way that the marking is not visible above the floor support.**