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**FI**

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**EN**

*\* Store the manual for future reference \**

**336155, 336150 & 336160 Portable toilet raisers**  
**336180 Height-adjustable toilet raiser with arm rests**

***Read this manual carefully and follow the assembly instructions.  
Careful assembly guarantees safe use.***

Do not use this equipment without first reading and understanding this instruction manual. If you are unable to understand the warnings, cautions and instructions, contact a healthcare professional, dealer or technical personnel before attempting to install or use this equipment - otherwise, injury or damage may occur. Each individual should always consult with their physician or therapist to determine the best suited assistive aid for them and the proper adjustment and usage of it.

*\* Please note that the illustrations in this manual are only for guidance and may contain differences with the actual product.*

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**Se installationsfilmer på YouTube:** →

**General safety instructions:**

- Avoid putting your fingers or other parts of the body in places where they might get pinched
  - Operating temperature between -30°C and 40°C (product that has been in a very hot or cold place should be handled carefully so as not to injure yourself)
  - Do not make structural adjustments or changes!
  - Regularly check the general condition of the product and the tightness of the fasteners. **A defective device must not be used.**
- Don't try to fix it yourself but take it to a professional!

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**Watch installation videos on YouTube:** →

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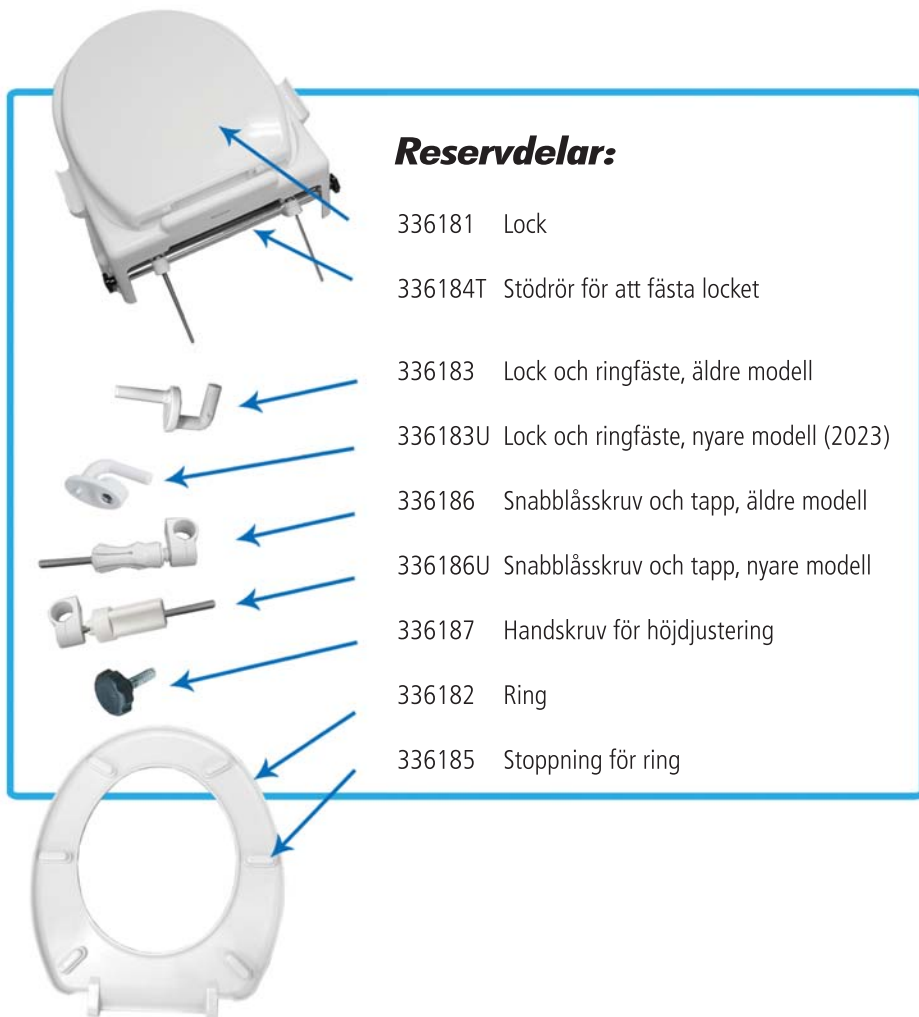
**Technical information:**

**Intended use:** The toilet raiser assists the user while using the toilet. Raiser can be used to raise the height of the toilet seat, which makes it easier to get on and off the toilet.

- Height: 5 cm, 10 cm or 15 cm
- Maximum user weight: 150 kg
- Dimensions: 40.7 cm x 36.6 cm
- Opening size: 26 x 22 cm
- Non-scratch mounting brackets
- Ultrasonically welded seamless and dirt-repellent surface
- Front edge support prevents tilting
- Indentations in the front and back
- Suitable for most toilet seat models
- Installation without tools

**Installation:**

Remove the ring and the lid of the toilet seat (for most models, the release is done by lifting). If the toilet raiser can be installed permanently without removing the ring and lid, they can also be left in place. In this case, lift them against the back of the seat before installation.

Attach the mounting brackets to the sides of the toilet raiser as shown in the picture. Pad is fitted so that it is placed in the recess at the bottom of the toilet raiser and press lightly. Turn the fastening screw so that it moves inwards.

Place the raiser on the toilet seat. Tighten the screws on the mounting brackets so that the pads are pressed against the outer edge of the toilet seat and the bracket is in place.

Check that the raiser is firmly in place. The toilet raiser is ready for use.

**Loosening and securing the toilet raiser is done by loosening or tightening the screws on the mounting brackets.**

**Removing the toilet raiser:**

Loosen the mounting pads by twisting the screws. Lift the toilet raiser out of the toilet seat.

**Washing:**

Clean by wiping with a damp cloth and a mild detergent. The product can also be sprayed clean with water. To remove difficult stains, it is recommended to use grease-dissolving cleaning agents, pH value 5-9 or 70% disinfectant. If necessary, the product can be immersed in water and/or sterilized in an autoclave at 85 degrees for a maximum of 1 minute (without mounting brackets).

**Technical information:**

**Intended use:** The toilet raiser assists the user while using the toilet. Raiser can be used to raise the height of the toilet seat, which makes it easier to get on and off the toilet. Armrests increase safety and make it easier to get up.

- Height adjustable without tools: 8 cm, 11 cm or 15 cm
- Seat ring size: 36 x 38 cm | Opening size: 23 x 27 cm
- The space between the armrests: min. 41 cm, max. 47 cm
- Dimensions without armrests: 44 x 18 x 48 cm
- Height with armrests installed: 37 cm
- Width with armrests installed: 56 cm
- User's max. weight: 130 kg
- Impact-resistant polypropylene
- Armrests can be turned up and/or removed
- Indentations in the front and back make washing easier
- Installation without tools

**Installation:**

The toilet raiser is supplied in its package pre-assembled, only the armrests need to be attached into their places. The quick lock fasteners are attached to the toilet raiser.

Remove the ring and the lid of the toilet seat (on most models, the release takes place by lifting).

Remove the attached quick lock fasteners by pulling them away from their pins (fig. 1). The height is adjusted using the black handwheel screws (fig. 2).

Remove the screws from the rear tube on both sides of the raiser. Remove the white rubber plug and place the tube at the desired height and tighten the screws back inside the tube (fig. 3).

Push the rubber plugs in place in the holes left empty (fig. 4).

Adjust the height in the same way also from the front.

fig. 2

fig. 3

fig. 4

fig. 1

Install the armrests by lifting their ends into the holes in the raiser. At the same time, press the quick release pin inside the tube of the armrest so that it goes into place, causing the quick release to click to locked position (fig. 5).

fig. 5

Adjust the mounting rods to a suitable distance from each other so that they fit into the holes of your toilet seat (fig. 6).

fig. 6

Place the riser on to your toilet seat so that the mounting rods are visible from below and are straight.

fig. 7

First, place the smaller part of the quick lock fastener on the mounting rod with the hat side forward and press it against the bottom of the toilet seat. Then push the larger part of the fastener into the rod with the narrow end first, all the way inside the smaller part and lock it by twisting (fig. 7).

fig. 8

Check that the fasteners are firmly in place (fig. 8). The toilet raiser is now ready for use. If you wish, you can lift the armrests out of the way or remove them completely.

**Removing the toilet raiser:**

Unscrew the quick lock fastener until you get the larger part out of the smaller one. The fastener will release, and you can pull it off the mounting rod. Lift the raiser off the toilet seat.

**Washing:**

Clean by wiping with a damp cloth and a mild detergent. The product can also be sprayed clean with water. To remove difficult stains, it is recommended to use grease-dissolving cleaning agents, pH value 5-9 or 70% disinfectant. If necessary, the product can be immersed in water and/or sterilized in an autoclave at 85 degrees for a maximum of 1 minute.

**Tukimet Oy**  
**Kaivopuistontie 33**  
**26100 RAUMA**  
**FINLAND**  
**[www.tukimet.fi](http://www.tukimet.fi)**

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Copyright Tukimet Oy